



# Vibby

ATE  
**urmet**

## The remote control with a built-in fall sensor



Falls at home can happen at any time of the day and are the leading cause of accidental death among people over 65.

- It has been estimated that one hour without assistance can result in:
- Higher rates of hospital admission
- Psychological and physical consequences
- Increased healthcare costs

**Vibby** is an automatic fall detector that can be worn on the wrist or around the neck, helping older adults stay independent and safe at home.

When Vibby detects a fall, it sends a coded alert to the operations center through the **MyoSOS** telecare terminal.



With an 87% fall detection rate, Vibby is an effective solution to reduce response and assistance times, ensuring safety for the user and peace of mind for the whole family.



Compatible  
with  
**myoSOS**



**Manual or automatic  
alarm deactivation**  
By reducing the number of false alarms, users are encouraged to wear the device.



**Battery life  
of up to 2 years**  
Cuts down on technician visits and maintenance operations.



**Customizable**  
Can be worn as a bracelet or as a pendant.



# Vibby



## Features

Product code	> 1-3915
Manual alarm	> By pressing the SOS button in the center of Vibby, the user can manually trigger an alarm.
Automatic alarm	> A set of sensors and an advanced algorithm automatically trigger an alarm in the event of a severe fall.
Alarm cancellation	> If the user stands up within 30 seconds after the fall and remains upright, the alarm is automatically canceled.  > Manual cancellation with sensitive sensor: the alarm can be canceled by covering Vibby with the palm of the hand for a few seconds.
Technical specifications	> Water resistance (IP67 standard) > Shock resistance: IK04 > Cleaning: water and soap only > Length: 37 mm > Width: 33 mm > Thickness: 12.7 mm > Weight: 35 g > Hypoallergenic plastic > Range: 300 meters in open area
Battery	> Battery: CR2477 lithium / LiMnO2 > Battery life: up to 3 years > Battery replaced by technician.
Monitoring	> 24-hour survival check > Battery level monitoring.
Compatibility	> Compatible with MyoSOS telecare terminals, codes: PA142, PA148.

